


tonen™
Customizable Hearing Aid
Model: HA-1



Quickstart
Set up for the hearing aids and the app

Included in the Box

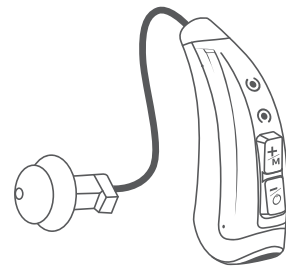

Hearing Aid




Hearing Aid



Charging Case



3 Sets of Ear Tips

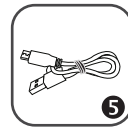

USB Cable



Cleaning Tool




Wax Guards


www.tonenhearing.com

Turning ON and OFF

Charging Case

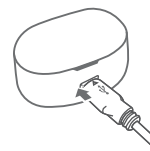


Micro-USB to case, USB to external power supply

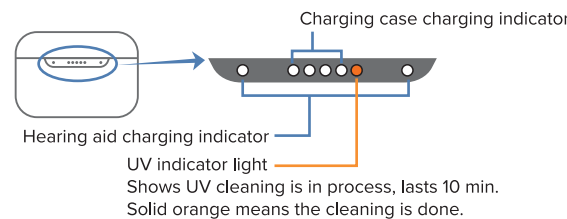


Indicator lights on the case

Remove the hearing aids from the charging case to turn ON automatically. Return to case to turn OFF.

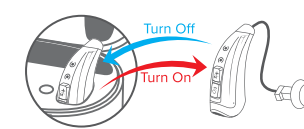

Press and hold buttons on the hearing aids for 3 seconds to turn them ON/OFF.

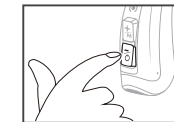


Turning ON - The hearing aids will automatically go into Bluetooth Pairing mode, with the lights blinking Blue and White.
Turning OFF - The hearing aids will automatically go into charging mode, with the charging case lights blinking Blue.

Bluetooth pairing to your phone

Before you can access the features that follow, your phone needs to be paired with the hearing aids.


For iPhone

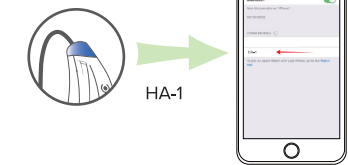

- In your phone's Settings menu, open the Bluetooth setting.
- Make sure Bluetooth is turned on.
- Find HA-1 under Other Devices and select it to pair.
- Tap on Pair.

Bluetooth pairing to your phone

For Android

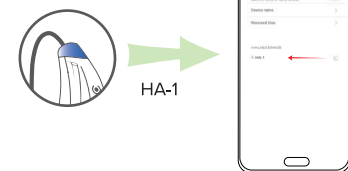

- In your phone's Settings menu, open the Bluetooth setting.
- Make sure Bluetooth is turned on.
- Click on Pair New Device
- Find HA-1 under Available Devices and select it to pair.

Download the Audiologist App


The app is compatible with Android and iOS. You can personalize your hearing experience. Download the free app 'Audiologist' from the App Store or Google Play Store.



For more details, please read the enclosed User Guide.

Connecting Hearing Aids to Audiologist App

- In the Audiologist App, press "Connect Aids" under the Home tab at the bottom of the screen.

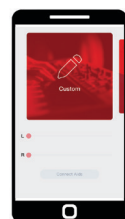


- Press the device name "HA-1-L" and "HA-2-R". You are fully connected once both hearing aids say "Connected" in the app.
- You can disconnect the hearing aids in the app by going to "Connect Aids" and swiping left on "HA-1-L" and "HA-2-R" and pressing "Disconnect".



Taking A Hearing Test

- In the Audiologist App, press the "Test" tab at the bottom of the screen.
- Press "Hearing Test" and go into a quiet room. Have hearing aids in your ears.
- Follow the instructions within the app. Take the test to your best ability for the most accurate results.

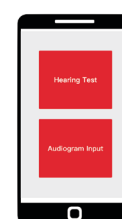

Taking and Making Calls with Bluetooth



	- 1 press to turn up volume.	
	- 1 press to answer calls. - Press and hold to reject/end calls. - 1 press to turn down volume.	

Listening to Music



	- 1 press to turn up volume. - 3x press while music is playing for previous song.	
	- 1 press to turn down volume. - 2x press to play/pause. - 3x press for next song.	

Activate Smartphone Voice Assistant



	3x press to activate smartphone voice assistant. Note: to use voice assistant, music/phone calls must be paused.	
	2x press to activate music player.	